INSTRUCTIONS FOR USE

- Break the sterility guarantee seal just before use by turning the two parts of the protective sleeve in opposite directions.
- Screw the pre-threaded needle onto the pre-threaded hub of the syringe.
- Remove the plastic cap just before the injection is administered. It can be used again to remove the needle from the syringe to avoid risk of needle stick injury.
- Discard the used needle into a puncture-resistant container in accordance with all applicable medical waste regulations.

WARNINGS

- Never recap needles by hand. Use only accepted dental recapping procedures. Recapping needles by hand or recapping bent or broken needles can cause unintended needle sticks.
- Do not bend, break, or otherwise stress needles as serious injuries to you and/or your patient can occur.
- Do not insert needle to hub during injections, as needles can break and become lodged in patient's tissue, potentially causing serious permanent injury. Do not use short needles (< 30 mm) when the expected depth in soft tissue approaches the length of the needle.
- Avoid extreme pressure and excessive needle movement during injection as this may cause needles to break, which may result in serious injury to you and/or your patient.
- Be especially vigilant when repositioning a needle in a patient who appears to be apprehensive.

CAUTIONS

- Store in original packaging until ready to use.
- Read the Instructions For Use supplied with your injection device.
- If seal is broken, the needle is NOT STERILE and MUST NOT BE USED.
- For single use only: This device is for use in a single patient. Any reuse exposes other patients and the user to critical risks, including infection and trauma since the needle is no longer sterile and is more likely to break."
- If multiple injections are required, replace the needle for each cartridge in order to minimize the risk of needle point damage.

1 patient / 1 cartridge / 1 needle

PRESENTATION

100 two-point disposable non-resterilisable dental needles

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